

**Power Eyes from HTE.**  
**Oxygen Massager for Eyes**  
**with Far Infrared.**




**The Power Eyes** combines air pressure, heat compress, and vibration massage which can improve blood circulation around the eye area. This deep therapy helps speed up the metabolism process around the eyes. It also prevents eye disease, aging, and also helps eliminate dark circles and fine wrinkles around the eyes. In addition, it can also help you obtain a good quality night of sleep.

**Who Does Power Eyes Help Specifically?**


1. **The Elderly:** Relieve eyestrain, dry eyes, watery eyes, blurry vision and discomfort to light due to gradual vision degeneration. May improve age related loss of elasticity of the eyes (presbyopia).
2. **Computer Users:** Relieve eye strain, eye pain and blurry vision. Helps keep eyes moist and may inhibit computer vision syndrome.
3. **Drivers:** Dispels visual fatigue caused by long periods of driving which can result in dry eyes, swelling, stiff shoulders and headache.
4. **Contact Lens Users:** Soothes dry, achy and inflamed eyes after wearing contact lenses for long periods of time.
5. **Adults:** Increase blood circulation and prevents dark circles and wrinkles around the eyes.
6. **Teenagers:** Alleviates eye fatigue caused by extended reading and writing. and may help prevent or reduce nearsightedness.

## Eye Powers Features:

1. **Power Eyes Eyeshade:**  
Auto - adjustment on eyeshade to the shape of the face (thin, wide, short, and long).
2. **Internal Low Pressure Air Bag:**  
Safe and quiet low pressure air pump massages the eyes without pushing down on them.
3. **Micro-Computer Digital Programming:** Precise micro-computer programming allows for 15,10, and 5 minute intervals for use.
4. **Nano - Energy Eyeshade:**  
Precise anti-bacterial Far Infrared heat and anions.
5. **Air Pressure + Thermal Heat:**  
Option to select different seismic vibration modes- Rub, Pich, Press, & Auto (combination of all 3) with Air Pressure and Heath (starting from 104 °F to 109 °F).
6. Close mask body on your face, hold strap with both hands behind your head and adjust as needed.

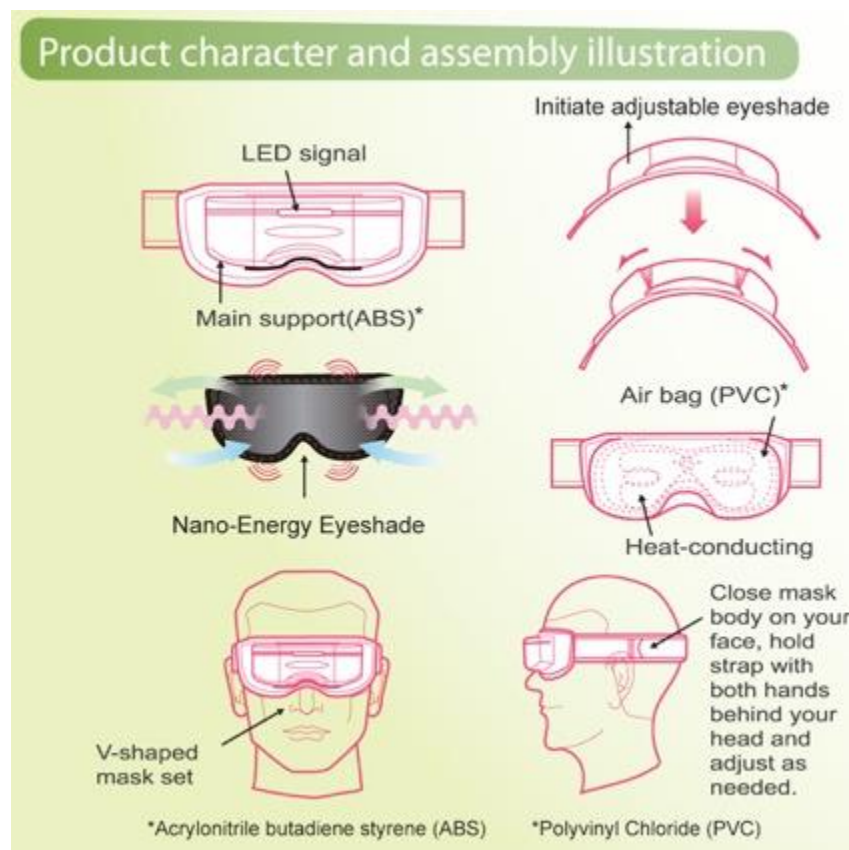

Here are some quick reference guides for Power Eyes Operation:

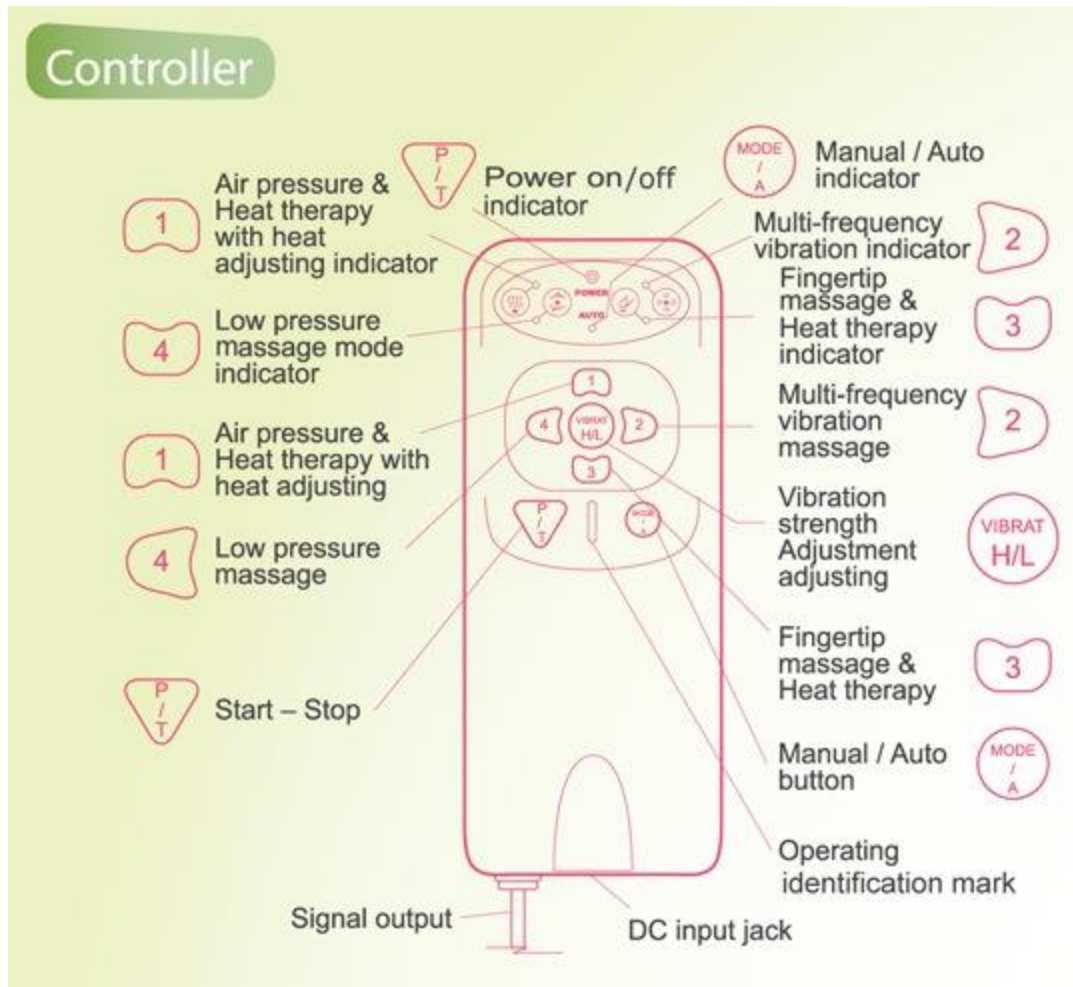

## Quick reference guide



**Start/Stop Button:** Press the key once to turn on the machine and it will run for 15 minutes (1 beep). Press the key again and it will run for 10 minutes (2 beeps). Press the key again and it will run for 5 minutes (3 beeps). Press the key again to turn the machine off (long beep).



**Manual / Auto Button:**

※ Press  
1 beep / Manual  
2 beeps / Auto



**Air Pressure & Heat therapy with heat adjusting button : (3 settings)**

※ Press  
1 beep / low temperature & low pressure;  
2 beeps / medium temperature & medium pressure;  
3 beeps / high temperature & high pressure



**Vibration strength adjustment button : (3 settings)**

※ Press  
1 beep / weak vibration;  
2 beeps / medium vibration;  
3 beeps / forceful vibration



**Multi-frequency vibration massage button**





**Fingertip massage & Heat therapy button**




**Low pressure air massage button**




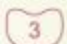

## Auto / Manual Operating Steps

### ■ Auto start:

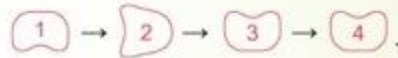

1. Close your eyes and put the eye mask on (using the operation identifying mark to be the point mark)
2. Turn start-stop button on [  ]. (in accordance with default function "15" minutes, or choose the proper periodic term of 15, 10, or 05 minutes.)
3. Press Manual / Auto button [  ] Press once, and you'll hear 2 beeps that will start the Auto Mode (this will activate the default massage sequence of



### ■ Manual Operation Steps:

1. Close your eyes and put the eye mask on.
2. Turn the Start/Stop button on [  ]. for 15, 10, or 05 minute periods.
3. Press one of the buttons below and it will start by your choice (  →  →  →  )

### ★ Automatic and Manual Operations: A Detailed Explanation

1. Auto to Manual  
While operating in auto-mode, press the [  ] key and you'll hear 1 beep. This will cancel the automatic mode. Then, choose another function key to change to the manual-mode.
2. Manual to Auto  
While operating in manual mode, press the [  ] key and listen for 2 beeps. This will allow the change from manual to auto-mode.

## Power Eyes Specifications:

1. Far Infrared: 8-12 microns.
2. Power usage: Use AC 110V or DC 6V - "AA" Batteries x 4 = 6V
3. Includes: Power saving mode operates by computer micro-chip. Automatic turn off function. It will beep when the battery is low.
4. **Caution: Do NOT use this product if:**
  - a. experiencing various symptoms of eye disease, such as glaucoma, cataracts or a detached retina, etc.
  - b. you have had an eye operation in the past six month.
  - c. if any concerns, please consult with your eye doctor before using.

## POWER EYES AWARDS

1999 Silver Medal - Geneva, Switzerland, International invention exhibition.  
2005 Gold Medal - Beijing, China, International technology exhibition.  
2005 Gold Medal - Taipei, Taiwan, International invention show.  
2006 Bronze Medal - Taipei, Taiwan, International invention show.  
2007 Bronze Medal - Taipei, Taiwan, International invention show.  
2008 Silver Medal - Malaysia International technology exhibition.  
2008 Gold Medal - Russia International Invention show.

## PRICE POWER EYES:



**Warranty: 1 year.**

 **US\$149.00**

 **CA\$229.00**

14 day **FREE** trial from day of delivery.

**TO ORDER CALL (940) 761-4045**

**ADVANCED CLINICAL NUTRITION**

**Dr. Donna F. Smith**

**[www.AdvancedClinicalNutrition.com](http://www.AdvancedClinicalNutrition.com)**